



virox

CLOPREP Applicator

Swiss Chemi Virox CLORPREP Applicator is a tinted solution of 2% w/v ChlorhexidineGluconate (CHG) and 70% v/v Isopropyl Alcohol (IPA). It is indicated for skin antiseptics prior to invasive procedures, this product helps reduce bacteria on skin to diminish the risk of surgical site infection.

Indicated for skin antiseptics prior to invasive procedures, this product helps reduce bacteria on skin to diminish the risk of surgical site infection.

For Hospital and Healthcare professional use only
Antimicrobial persistence of 72 hours.
Latex Free

When Applying Solution:

completely wet the treatment area with antiseptic on dry surgical sites (e.g., abdomen or arm): use repeated back-and-forth strokes for 30 seconds moist surgical sites (e.g., inguinal fold): use repeated back-and-forth strokes for 2 minutes maximal treatment area for one applicator is approximately (13 in. x 13 in. (178.8 in 2)). do not allow solution to pool; tuck prep towels to absorb solution, and then remove avoid getting solution into hairy areas. Wet hair is flammable. Hair may take up to 1 hour to dry. when prepping skin folds, toes, or fingers, use a sterile-gloved hand to hold skin apart until completely dry. Otherwise, skin may adhere to itself

Composition:

2% Chlorhexidene Gluconate Solution
70% Isopropyl Alcohol v/v

Ordering code:

SCVCPA035 Virox Clorprep Applicator 3.5 ML
SCVCPA105 Virox Clorprep Applicator 10.5 ML
SCVCPA260 Virox Clorperero Applicator 26.0 ML

Special Advice

- > For external use only, Do not swallow
- > Flammable. Keep Away from fire and flame
- > Avoid Contact with eyes.
- > Discontinue use if skin irritation or redness develops
- > Do not use after expiry
- > Contraindicated if case of know hypersensitivity to CHG.

